



Surgical Female Pelvic Trainer (SFPT) with Advanced Surgical Uterus

Part No: **60283**

An advanced model for learning a wide range of laparoscopic uterine surgical techniques and procedures.

Skills

- The Advanced Surgical Uterus contains a number of pathologies providing for the removal of:
 - 2 myomae
 - 1 ovarian cyst
 - 1 ectopic pregnancy
- Laparoscopically assisted vaginal hysterectomy (LAVH)

Features

- The Advanced Surgical Uterus:
 - has highly realistic and accurate anatomy including fluid filled vessels
 - provides lifelike visualisation and instrument response
 - is a replaceable item once pathologies have been removed and hysterectomy completed
- Insufflation is represented by the skin support hoops under the abdominal wall
- Provides practice and role playing for gynaecological perioperative teams
- Includes a realistic vaginal canal and cervix for inspection, in which a uterine manipulator can be used

Package supplied

- 1 base
- 1 Advanced Surgical Uterus **60287**
- 2 Skin Support Hoops **60284**
- 1 Insufflated Female Abdominal Wall **60265**
- 1 Female Perineum & Bowel **50119**



Far left: Abdominal wall removed showing Advanced Surgical Uterus and Skin Support Hoops in place

Left: Trainer with abdominal wall secured in place



Advanced Surgical Uterus

Part No: **60287**

Contains:
2 myomae
1 ovarian cyst
1 ectopic pregnancy



Female Perineum & Bowel

Part No: **50119**

Helix Connectors

Part No: **90100**

Insufflated Female Abdominal Wall

Part No: **60265**

Skin Support Hoops

Part No: **60284**

Care of products made from latex foam rubber

- Latex foam is a natural product; store away from strong light and in an airtight bag to reduce exposure to ozone. Ozone is generated by some electrical equipment including telephones and computers.
- Wash your hands before touching the foam. (If the foam is handled after contact with certain metals such as copper, it becomes yellow and discoloured).
- The foam may be washed effectively using any soap and rinsed afterwards. 'Vanish'™ soap is very effective.
- Do not ingest the product and practice normal hygiene after handling the products.



Designed and manufactured by Limbs & Things Limited,
Sussex Street, St. Philips, Bristol, BS2 0RA, UK
Telephone: +44 (0)117 311 0500 Fax: +44 (0)117 311 0501
E-mail: sales@limbsandthings.com www.limbsandthings.com

Attaching the Advanced Surgical Uterus

1. The Uterus is mounted onto the back of the perineum using the small flanges covered with velcro. Press the flanges onto the receiving velcro patches on the back of the perineum. Ensure that the distal end of the vagina is fed through the opening in the perineum **fig 1**.

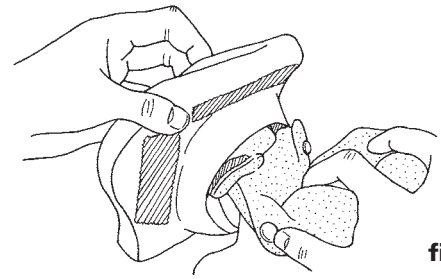


fig 1

2. The uterus, ligaments and bowel are fed through the inferior opening of the pelvic shell **fig 2**.

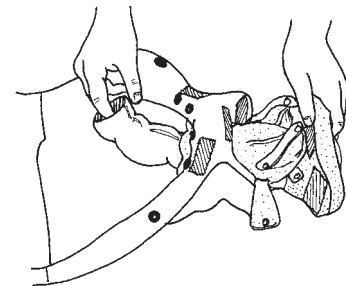


fig 2

3. Pegs securing the vaginal ligaments are placed into the corresponding grommets laterally from the vagina **fig 3**.

Grommet Holes: 30 & 31 (See Grommet Hole positions sheet)

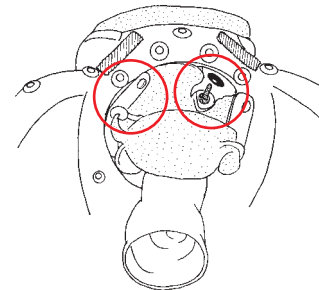


fig 3

4. Peg down the utero-sacral ligaments either side of the bowel **fig 4**.

Grommet Holes: 23 & 24

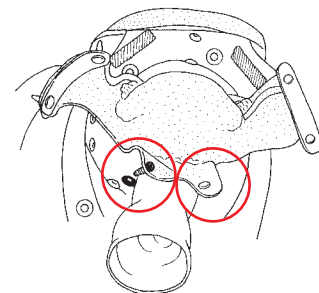


fig 4

5. Press the pegs into position which will secure the broad ligament **fig 5**.

Grommet Holes: 19 & 27, 20 & 29

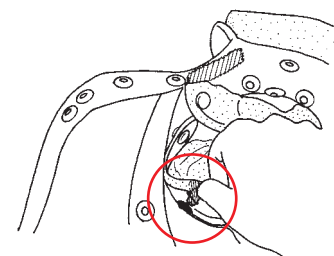


fig 5

6. Press the pegs into position which will secure the round ligaments **fig 6**.

Grommet Holes: 32 & 36

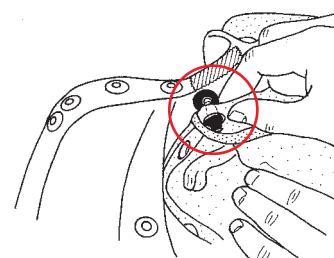


fig 6